# schülke -}



Ready to use disinfectant wipes impregnated with an alcoholic solution.

# mikrozid<sup>®</sup> AF wipes

### **Our Plus**

- ready to use, impregnated disinfection wipes
- very broad biocidal activity within the shortest possible time
- dries rapidly
- wide range of wipe sizes and packaging

# **Application areas**

Alcohol-based rapid disinfection of medical devices in all areas and all kinds of wipeable surfaces with an increased risk of infection and where short contact times are required, e.g.:

- Patient nearby surfaces
- Patient treatment units
- Examination couches
- Operating tables and nearby work surfaces
- Surfaces of medical equipment

### Instructions for use

Thoroughly wipe the surface with the impregnated wipe and allow to take effect. Make sure to wet surfaces completely and keep them wet for the whole exposure time. Make sure that all visible dirt is removed prior to disinfection. When opening the lid and inserting the first cloth, make sure that hygiene requirements are maintained. Make sure to close the lid tightly after use. Do not treat surfaces that are sensitive to alcohol, e.g. acrylic glass. Not for final disinfection of semicritical and critical medical devices!

# **Microbiological efficacy**

Efficacy	Concentration	Contact time
bactericidal EN13727, EN16615, in accordance with VAH Guidelines with short contact times - dirty conditions	ready-to-use	1 min.

Efficacy	Concentration	Contact time
tuberculocidal EN14348 - dirty conditions	ready-to-use	1 min.
levurocidal EN13727, EN16615, in accordance with VAH Guidelines with short contact times - dirty conditions	ready-to-use	1 min.
fungicidal EN13624, EN13697 - dirty conditions	ready-to-use	2 min.
virucidal against enveloped viruses in accordance with DVV (German Registered Association for Combating Viral Diseases)//RKI Guideline	ready-to-use	30 sec.
Norovirus EN14476 - dirty conditions	ready-to-use	1 min.
Polyoma SV40 in accordance with DVV (German Registered Association for Combating Viral Diseases)//RKI Guideline	ready-to-use	10 min.
Rotavirus in accordance with DVV	ready-to-use	30 sec.
Adenovirus (type 5) in accordance with DVV (German Registered Association for Combating Viral Diseases)//RKI Guideline	ready-to-use	5 min.

### Certificates

- ÖGHMP certificate
- VAH certificate
- IHO listed



# mikrozid<sup>®</sup> AF wipes

# **Product data**

#### Composition:

100g agent solution contain the following active ingredients: 25 g Ethanol (94 %), 35 g Propan-1-ol

#### Chemical-physical data

Color	colourless
Density	ca. 0,89 g/cm3 / 20 °C
Flash point	27 °C / Method : DIN 51755 Part 1
Form	liquid
рН	Not applicable
Viscosity, dynamic	not determined

# **Special advice**

# Use disinfectants safely. Always read the label and product information before use.

The product is highly compatible with metals and synthetics (except acrylic glass and alcohol-sensitive varnishes). Please follow carefully any national fire and explosion protection guidelines when using alcohol-based disinfectants. Only treat small surfaces. The wiping range depends on the room temperature and on the structure of the surface to be disinfected. More information avaible on request. **Refilling of the tin** 

First disinfect hands and wear new protective gloves. During the refilling process, please ensure that the gloves are not contaminated by other activities. On every refill, reprocess the lid closure. In reprocessing, subject the lid closure to disinfectant cleaning with an alcohol based surface disinfectant.

Following sizes are available:

- mikrozid® AF wipes (10 x 150 wipes 14 x 18 cm) Surface couverage\*: approx. 0.3 m<sup>2</sup>
- mikrozid<sup>®</sup> AF wipes Refill (20 x 150 wipes 14 x 18 cm) Surface couverage\*: approx. 0.3 m<sup>2</sup>
- mikrozid<sup>®</sup> AF wipes Jumbo (10 x 200 wipes 20 x 27 cm) Surface couverage\*: approx. 0.5 m<sup>2</sup>
- mikrozid<sup>®</sup> AF wipes Jumbo Refill (12 x 200 wipes 20 x 27 cm) Surface couverage\*: approx. 0.5 m<sup>2</sup>
- mikrozid<sup>®</sup> AF wipes *premium* (12 x 50 wipes 20 x 20 cm) Surface couverage\*: approx. 0.5 m<sup>2</sup>

schülke guarantees, if used in the correct way, a shelf life of 3 months after the package has been opened.

# Information for order

Item	Delivery form	ltem no.
mikrozid® AF wipes premium softpack (50 wipes)	12/Carton	on request
mikrozid® AF wipes Jumbo Refill (200 wipes)	12/Carton	on request
mikrozid® AF wipes Jumbo tin (10 x 200 wipes)	10/Carton	on request
mikrozid® AF wipes tin (10 x 150 wipes)	10/Carton	on request
mikrozid® AF wipes Jumbo refill (12 x 200 wipes)	1/Carton	on request
mikrozid® AF wipes refill (20 x 150 wipes)	1/Carton	on request

These products are not available in every country. For more information please contact our local subsidiary or distributor.

# Accessories

Accessories	ltem no.
wall bracket Jumbo tub	134421
Wall mounting for the dispenser (small)	134417

# **Related Products**

- mikrozid<sup>®</sup> PAA wipes
- mikrozid<sup>®</sup> sensitive wipes
- mikrozid<sup>®</sup> universal wipes premium

# **Environmental information**

schülke manufactures products economically and with advanced, safe and environmentally friendly production processes while at the same time maintaining out high quality standards.

# **Expert opinion and information**

Please visit our website for an overview of all available literature/reports on the product: www.schuelke.com. For individual questions: Customer Sales Service Phone: +49 40 52100-666 E-Mail: info@schuelke.com

\* The surface couverage of one wipe depends on the surrounding conditions and the structure of the wiped surface.



Schülke & Mayr GmbH holds a Manufacturer's Authorisation according to sect 13 para 1 German Drug Law and Certificates of GMP Compliance for medicinal products.



schülke Headquarters Schülke & Mayr GmbH Robert-Koch-Str. 2 22851 Norderstedt, Germany Phone +49 40 - 52100 - 0 Fax +49 40 - 52100 - 318 www.schuelke.com info@schuelke.com